

# LEMEX

## CLEANER/REODORANT

### AUSTRALIAN MEDICAL DEVICE CLASS 1

#### ARTG NO. 125529



**BENEFITS:**

- Inexpensive, general purpose detergent
- Fresh long lasting perfume
- pH Neutral - Excellent Materials Compatibility
- Non-corrosive
- Class 1 Medical Device

LEMEX is an inexpensive lemon perfumed detergent suitable for general use as a cleaner or reodorant. LEMEX is suitable for surface cleaning and use on other Class 1 Medical Devices and has been entered onto the Australian Register of Therapeutic Goods (ARTG). Using LEMEX as directed ensures Healthcare facilities comply with current Healthcare regulations.

LEMEX is a neutral detergent containing a powerful cationic surfactant and a fresh long-lasting lemon perfume. Being neutral it is non-corrosive to most fibres and common hard surfaces including normal domestic, commercial and most hospital/medical flooring.

LEMEX can be used for general purpose cleaning throughout facilities on surfaces such as floors, benches and most hard surfaces. For the disinfection of environmental surfaces use VIRACLEAN or SURFEX. LEMEX can be used for cleaning Class 1 medical devices such as operating tables, theatre trolleys and lights and other equipment within a Healthcare Facility. When used on Class 1 Medical Devices LEMEX should be used as a cleaner only and subsequent disinfection, where required, should be done with a Low Level Instrument Disinfectant such as INSTRUMAX PINK or SURFEX.

**DIRECTIONS FOR USE**

	DILUTION IN POTABLE WATER	DILUTION IN AUTOSCRUBBER
<b>GENERAL CLEANING</b>		
Normal Cleaning:	25mL/L with potable water	15ml/L with potable water
Heavy Cleaning:	50mL/L with potable water	
Deodorising:	200mL/L with potable water	
<b>CLEANING MEDICAL DEVICES</b>		
No visible soiling:	25mL/L with potable water	
Soiled with blood or body fluids	50mL/L with potable water	

*\*Make Fresh Solutions Daily.*

**CHEMICAL COMPOSITION AND PROPERTIES:**

Appearance (concentrate):.....	Clear yellow liquid
Appearance (dilute solutions): .....	Clear, light yellow liquid
Composition: .....	Surfactant blend
Detergent System .....	Cationic
Odour .....	Lemon
pH (concentrate).....	6-8 [Neutral]
Solubility.....	Soluble
Rinsibility.....	Total
Abrasives.....	Nil
Hazardous Status.....	Non Hazardous
Volatility .....	Nil
Flammability .....	Nil
Dangerous Goods Rating.....	Nil
Shelf life .....	2 years

**SAFETY DATA:** Information is listed in the Safety Data Sheet.

**PACK SIZE:** 2 x 5 Litre  
15 Litre

**HEALTH AND SAFETY STATEMENT:**

There is no way of predicting an individual's response to cleaning or disinfecting products. All cleaning and disinfecting products have a potential to cause skin and other irritation in susceptible individuals. Direct physical contact with the skin should be avoided. If irritation does occur when using this product or any other cleaning/ disinfecting solution then the individual should be removed from direct contact with the solution until a satisfactory safe working environment for that individual is determined.

**ENVIRONMENTAL STATEMENT:**

Whiteley manufactures products that comply with Australian and International environmental policy. Whiteley aims to ensure that every product is 100% biodegradable, whilst achieving optimal performance and benefit to customers. All Whiteley products are supplied in 100% recyclable containers. Whiteley and its staff are fully committed to ensuring that our environment and those international environs in which our products are used are enhanced by the use of technology, and that these environments are preserved for the enjoyment of future generations.

© 2021 This document remains the property of Whiteley Corporation Pty Ltd.

Alterations are not permitted without prior written authorisation from Whiteley Corporation Pty Ltd.